

Glaukos® iAccess™ Trabecular Trephine INSTRUCTIONS FOR USE

DEVICE DESCRIPTION

The Glaukos® iAccess™ Trabecular Trephine is a sterile, single-use, disposable ophthalmic instrument.

Catalogue #	Description
AX1	Glaukos iAccess Trabecular Trephine

INDICATIONS FOR USE

The Glaukos iAccess Trabecular Trephine is intended for use in ophthalmic surgical procedures to manually cut trabecular meshwork (TM) in pediatric and adult patients.

INSTRUCTIONS FOR USE

- A. Inspect the device for damage and use care when removing from packaging and handling device. Trephine cutting edge must remain sharp and can be damaged if contacted. Tip protector should remain in place if the instrument is placed onto the sterile tray.
- B. Place a gonioscope onto the cornea and adjust the angle of the patient's head and surgical microscope as needed to visualize the TM.
- C. The anterior chamber should be inflated with viscoelastic when using the iAccess Trabecular Trephine.
- D. Insert the iAccess Trabecular Trephine into the eye through a temporal, clear corneal incision (≥ 1.5 mm) opposite the desired site of tissue perforation. Take care not to damage the corneal endothelium or iris, and prop open the incision as needed in order to safely pass the trephine into the eye.
- E. Advance the trephine tip to penetrate the TM at a location opposite the corneal incision. With the tip of the instrument contacting the TM directly over the Canal of Schlemm, apply forward pressure with slight rotation (if necessary) to cut the tissue. Cutting depth is limited by shoulder of support tube to prevent over insertion.
- F. Upon exposure of Schlemm's Canal, and depending on the pressure in the eye, blood may reflux into the anterior chamber from the collector channels. Viscoelastic or balanced salt solution (BSS) can be used to tamponade or clear blood to improve visualization.
- G. Multiple perforations of the TM may be achieved without removing the device from the eye.
- H. Irrigate the anterior chamber with (BSS) through the corneal wound to remove all viscoelastic. Press down on the posterior edge of the incision as needed to facilitate complete removal of viscoelastic.
- I. Inflate the anterior chamber with saline solution as needed to achieve normal physiologic pressure.
- J. Ensure that the corneal incision is sealed.
- K. Dispose of the device after use in accordance with facility standard operating procedures for sharps and medical waste.

WARNINGS / PRECAUTIONS

- For prescription use only.
- Physician training is required prior to use of the iAccess Trabecular Trephine, including intraoperative gonioscopy.
- Do not use the Glaukos iAccess Trabecular Trephine if there is poor visualization of angle structures. Improper visualization could result in damage to adjacent eye structures.
- Do not use the device if the Tyvek lid has been opened or the packaging appears damaged. In such cases, the sterility of the device may be compromised.
- Do not re-sterilize or reuse the device as this may result in infection and/or intraocular inflammation, as well as occurrence of potential postoperative adverse events.
- Used or unused product and packaging may be disposed of in accordance with facility procedures for handling Sharps and medical waste.

CONTRAINDICATIONS

The Glaukos iAccess Trabecular Trephine is contraindicated under the following circumstances or conditions:

- In eyes where there is poor visualization of angle structures.

HOW SUPPLIED

The Glaukos iAccess Trabecular Trephine is supplied sterile and in a Tyvek-sealed blister tray with removable Sharps protector attached. The device has been sterilized by gamma radiation. One device is supplied in each unit carton. Each unit is labeled with a lot number that is provided on both the tray lid and unit carton.

EXPIRATION DATE









The expiration date on the device package (tray lid) is the sterility expiration date. In addition, there is a sterility expiration date that is clearly indicated on the outside of the unit carton. Sterility is assured if the tray seal is not punctured or damaged until the expiration date. This device should not be used past the indicated sterility expiration date.

STORAGE REQUIREMENTS

Do not expose to direct sunlight, water, high temperature, and high humidity.

LABELING

The following symbols are used on the device packaging.

Symbol	Definition	Symbol	Definition
	Catalogue/Model Number		Do not use if package is damaged
	Lot Number		Sterilized by Gamma Irradiation
	Do not re-use		For prescription use only
	Use By (year-month-day)		Consult Instructions For Use.

Manufactured for:

Glaukos Corporation

229 Avenida Fabricante

San Clemente, CA 92672 U.S.A

Tel: +1.949.367.9600, Fax: +1.949.367.9984

www.glaukos.com

Glaukos® is a registered trademark of Glaukos Corporation. iAccess™ is a trademark of Glaukos Corporation.

CAUTION: Federal law restricts this device to sale by, or on the order of a physician.